

Standard Flat Model




Model: ENS-A

Features:

- Electric adjustable overall height
- Lithium battery and charger
- Hand controller
- Powder coated aluminum drop down revolving guardrail
- Anti-rust aircraft aluminum upper frame
- Castors with a central brake locking system
- Extendable flex drain pipe
- Soft vinyl liner in blue
- Pillow included

Specification:

- External Length: 224cm/88"
- Internal Length: 196cm/77"
- External Width: 86cm/34"
- Internal Width: 64cm/25"
- Height: 41-45cm/16"-17.7"~81cm-85cm/32"-33.5"
- Working Weight: 440lbs/200kg
- Product Weight: 176lbs/80kg
- Gross Weight: 187lbs/85kg
- Package Size: 81"/205cm x 30"/75cm x 21"/53cm

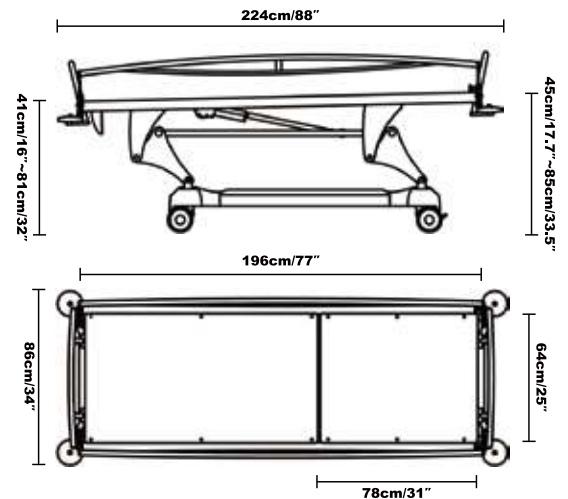

Tilting Backrest Model




Model: ENS-B

Features:

- Electric height adjustment
- Electric tilting backrest
- Lithium battery and charger
- Hand controller
- Powder coated aluminum drop down revolving guardrail
- Anti-rust aircraft aluminum upper frame
- Castors with a central brake locking system
- Extendable flex drain pipe
- Soft vinyl liner in blue
- Pillow included

Specification:

- External Length: 224cm/88"
- Internal Length: 196cm/77"
- External Width: 86cm/34"
- Internal Width: 64cm/25"
- Height: 41-45cm/16"-17.7"~81cm-85cm/32"-33.5"
- Working Weight: 440lbs/200kg
- Product Weight: 185lbs/84kg
- Gross Weight: 196lbs/89kg
- Package Size: 81"/205cm x 30"/75cm x 21"/53cm

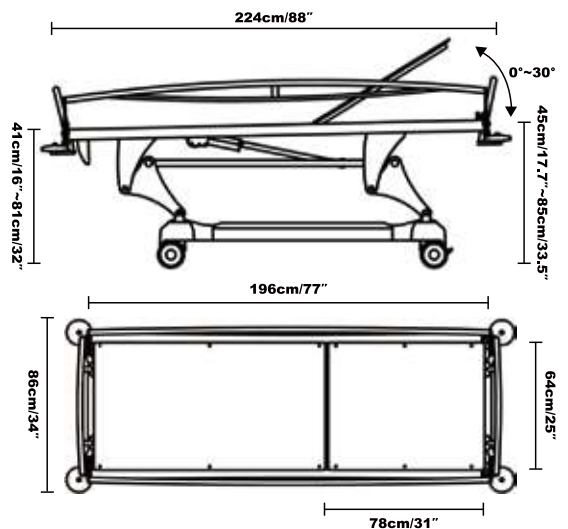

Middle Folding up Model





Model: ENS-C

Features:

- Electric height adjustment
- Electric middle folding up for turning over
- Lithium battery and charger
- Hand controller
- Powder coated aluminum drop down revolving guardrail
- Anti-rust aircraft aluminum upper frame
- Castors with a central brake locking system
- Extendable flex drain pipe
- Soft vinyl liner in blue
- Pillow included

Specification:

- External Length: 224cm/88"
- Internal Length: 196cm/77"
- External Width: 86cm/34"
- Internal Width: 64cm/25"
- Height: 41-45cm/16"-17.7"~81cm-85cm/32"-33.5"
- Working Weight: 440lbs/200kg
- Product Weight: 198lbs/90kg
- Gross Weight: 209lbs/95kg
- Package Size: 81"/205cm x 30"/75cm x 21"/53cm

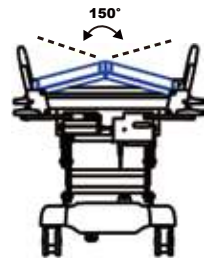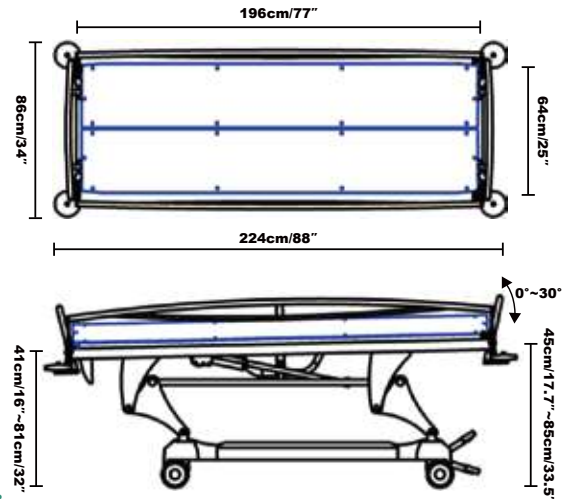

Overall Tilt Model




Model: ENS-D

Features:


- Electric height adjustment
- Electric overall tilting
- Lithium battery and charger
- Hand controller
- Powder coated aluminum drop down revolving guardrail
- Anti-rust aircraft aluminum upper frame
- Castors with a central brake locking system
- Extendable flex drain pipe
- Soft vinyl liner in blue
- Pillow included

Specification:

- External Length: 224cm/88"
- Internal Length: 196cm/77"
- External Width: 86cm/34"
- Internal Width: 64cm/25"
- Height: 41-45cm/16"-17.7"~81cm-85cm/32"-33.5"
- Working Weight: 440lbs/200kg
- Product Weight: 185lbs/84kg
- Gross Weight: 196lbs/89kg
- Package Size: 81"/205cm x 30"/75cm x 21"/53cm

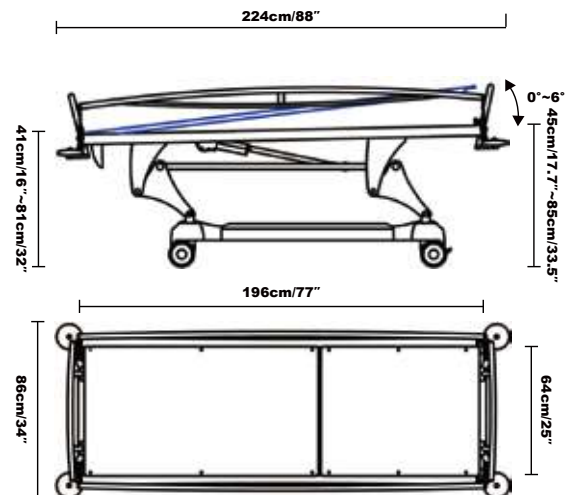

UNIVERSAL ALL WELDED MATTRESS FOR SHOWER TROLLEY

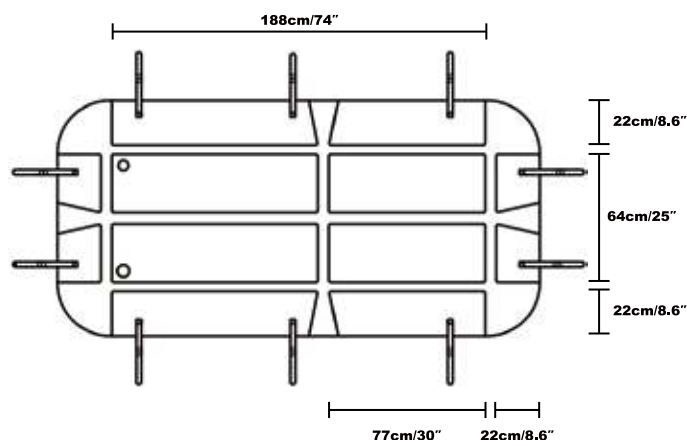

Model: NT01

Specification:

- Universal size, Suitable for all Noah Trolleys
- Soft vinyl liner in blue
- High density inner filler
- All-welded with sealed surface
- Thickness: 30mm
- Supplied with flexible drain hose



HEAVY DUTY CENTRAL LOCK CASTORS

Our medical 5-inch central control caster meets the European EN12531 medical standard, and the durable brake can be used more than 30000 times. Precise ball bearing is used in the castor center to effectively improve the durability and movement flexibility of caster. The soft surface of the wheel can effectively increase the grip, avoid trolley's sliding on the slippery bathroom floor, reduce the floor wear and effectively reduce the noise. The central brake locking system ensures that the wheels can remain stationary on a 15 degree slope.



TO LOCK & UNLOCK CASTORS

To lock the castors, press the right brake pedal and the four wheels will be locked simultaneously. When bathing the patient, we require that 4 casters must be locked to avoid side slip of the trolley. We must understand the importance of the stability of wheelbarrow, which can greatly improve work efficiency and avoid nurses' distraction while bathing. To unlock the caster, gently press the left pedal down with your foot. When the left pedal reaches the middle position, the right pedal will return to the middle position, and the four casters will be unlocked.



DIRECTIONAL MOVEMENT FUNCTION

When transferring the shower trolley for a long distance, it is recommended to fix the direction of one rear castor, which will help to control the moving direction of the shower trolley and avoid the side slippery. When the left pedal is pressed down, the right pedal will be lifted up automatically and the right rear wheel direction will be fixed.

Remove wheel direction restrictions, gently press the right pedal with your foot. When the right pedal reaches the middle position, the left pedal will return to the middle position, and the rear wheel direction restriction will be released.



Biocure Single Member P.C.
Epidavrou 2, Evosmos, 56224 Thessaloniki, Greece